

User's manual





こりつつしら

Item Product name / Model No.

Specification of charging

Basic description

Specification of charging	5Vdc, 1.0Adc
Power	Secondary battery embedded
Charging cable	Use USB C-JACK
Weight	100g
Size	72(L) x 40(W) x 72(H)mm
KC electromagnetic certificate	R-R-DLF-LF-A001D
Certificate No. of KC safety confirmation	XU101509-18001F (Secondary battery)
Manufacturer / Seller	DMLED Co., Ltd #409, TAWON TAKRA Knowledge Industrial Center, 176, Seonggok-ro, Danwon-gu, Ansan-si, Gyeonggi-do
Manufacturing country	Korea
Contact for A/S	+82-31-432-6769
Mode for use	3 steps (Blue/Yellow/Red)
Timer	1 time (Automatic power OFF at 5 min)
Time of charging	About 2.0 hours (May vary depending on charger)
Hours of consecutive use	About 1 hour and half minutes (About 15 times)
Environment of use	Room temperature (10°C~25°C)
Warranty period	1 year

Specification

5Vdc 10Adc

Litools Pet LED Bloc / LF-A001D

Product composition

Consumer Dispute Resolution Standard notified by Fair Trade Commission.

③ USB charging cable

Q

Main body

② User's manual

- นางจนร Q&A/ Frequently Asked Questions Is it possible to apply this device to sensitive pets? Possible; However, please refrain from applying this device to pets, in case that they are sensitive to light. Can it be applied to any dog, regardless of large or small ones?

LITOOLS PET LED BLOCK

User's manual

Possible; However, please again refrain from applying to the pets that are sensitive to light.

Is it possible to apply this device to cats?

cats as well that are sensitive to light.

exposure to their eyes.

Is this device waterproof?

Safety Certification

주시회사 디엠덴이디

R-R-DLF-LF-A001D

주식회사 디앤덴이디 / 한국

위 기자재는 「전파법」제58초의2 제3함에 따라 등록되었음을 증명합니다. It is verified that foregoing equipment has been registered under the Clause 3, Article 58-2 of Radio

국립전파연구원장 의자기

Director General of National Radio Research Agency

여성문의 방송등에게 4 대는 만든 안 " 기업 상징 가보자" 및 보안하여 유통하여나 합니다.

[Electromagnetic Certificate]

2020년(Year) 12월(Month) 02일(Day)

평용 전기정자기기

LF-A001D

상호 또는 성명

기자재명창(제품멸칭)

기기부호/추가 기기부호

Additional Fanishment and

기본모델명

파생모델병

등록번호

공복연원일 Bacof Registran

제조자/제조국가

Is it possible to apply this device to people? Yes, it can be applied to humans; Nevertheless, please avoid direct

Its waterproof function is limited; Therefore, please avoid high

Possible; However, please refrain from applying this device to the

- humidity or direct contact with water. Some colors of the light emitting surface have changed, is it okay?
- UV-A LED; However, there is no problem with product function.

There may be the discoloration of some light emitting parts, using

방송통신기자재등의 적합등록 필증 Registration of Broadcasting and Communication Equipments 안 전확인신고증명서 Confirmation Letter of Declaration

신교변호:

신교회사명: obrant 生:

제품명:

기본모델명:

903348

과생모듈병 (Series ModelX

公名列巻: RE SILENSES-ON

XU101509-18001F

803460

생제/안전기준상의 모델구분: 3.7 Vdr. 1 500 mAb

위 제품예만 해당한

비용의 정하에는 때를 병용에 따라 하기로 있습니다. 등을 받아서 입니다.

그 선원확인되고 대문에 연결 연중 (Derimine Status)

Dongguan UPine Electronic Technology Co., Ltd.

東京は本場: Dongguan UFine Electronic Technology Co., Lid 東本書を向 予念: Lunna Second Bridge, Tianshang Lake Villager Group, Lunna Village, Qiracsi Town, Dongguan, P.E.China

"보기용을 및 생활용을 반으면서당 시민구비는 배조비에만 같은 조 경기는 제요도배양 또는 배요도배양이 다구 선생하는 선생하는 선생이를 생긴다니다. We move that Conferentian Latter of Deckeration of the Subtry Conferentian for the show application in any with Article 2013, 2001, 2003 or 3400 of the Electrical Applicanes and Communic Products Select Control Act.

* 이 신고증업에는 '반각용을 및 생활용을 반찬본다면, 이 따른 제공의 단선성 화안에 환경된 것이라, 그 학의 다른 반물이 처음되는


[Battery KC Certificate]


필부서를 1. 전기용용의 인원환경부를 및 제공부족 (List of Critical Components)(전기용공에 한국하다) 기본보명 - 제상보명의 역용 Georgium of the lease and series modell

한국기계전기전자시험연구원장

Korea Testing Certification

Luoma Second Bridge, Tiansheng Lake Villager Group, Luoma Village, Gingxi Town, Dongguan, P.R.China

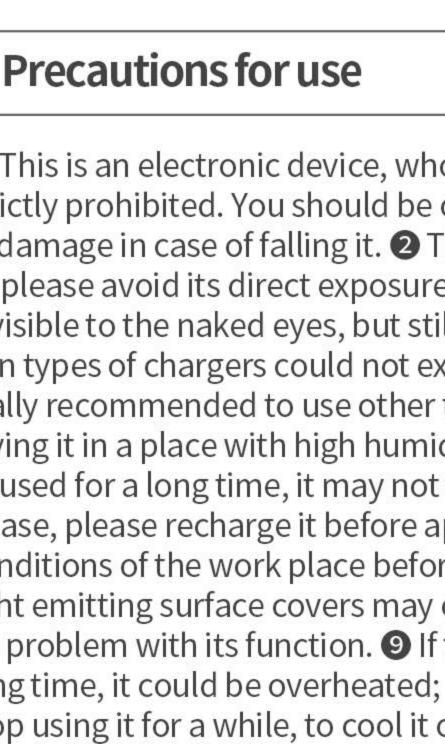

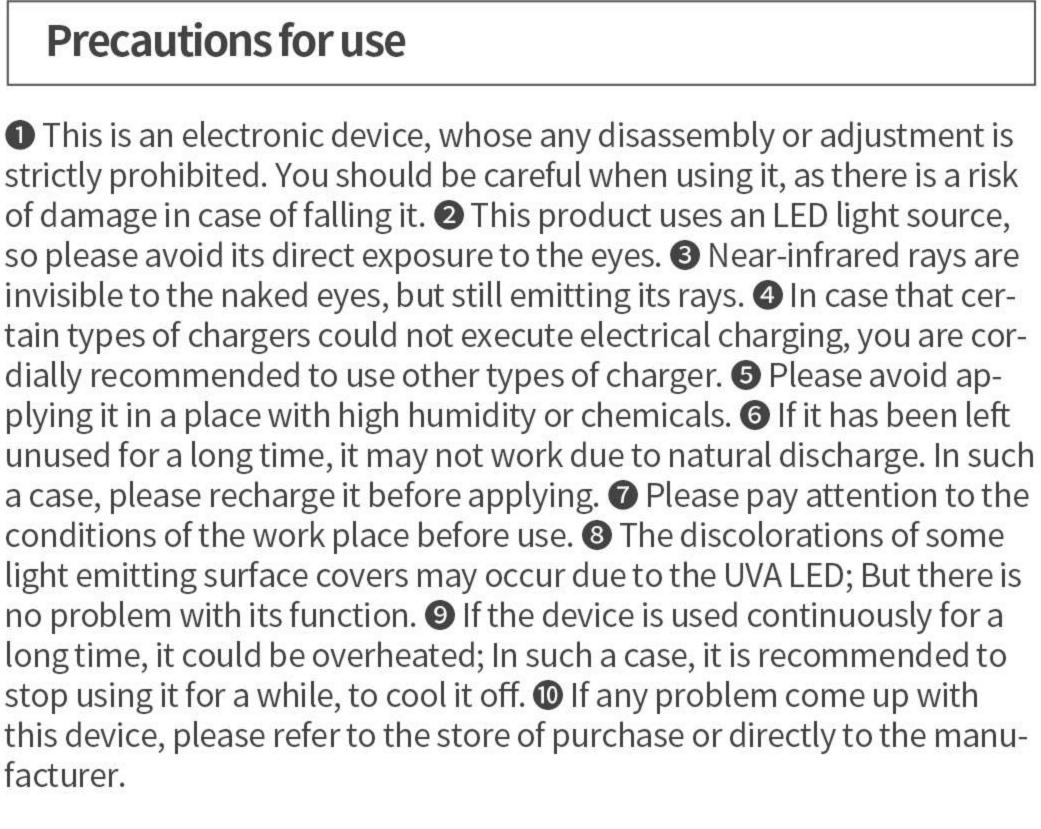


2 minute and 40 seconds passed

4 minute passed

5 minute passed

(Automatic power OFF)

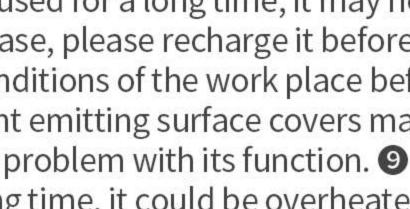

facturer.

How to use Intensive STEP 2. STEP 3. STEP 1 Start selection Press the power button It closely adheres and cares Press mode button to uncomfortable areas of to operate Mode 1. choose one among blue/yellow/red. companion animals. Indicator for hours of use Indication Number Hours of use 1 Unit ON 000 1 minute and 20 seconds passed <>

• 0 0

• 0

(h)	<>>
_ con	npleted – White
(h)	<>>

Product indicator shows

How to charge

Charging - Red

the hours of use.

※ Hours of use, can be used for about 15 times (It may) vary depending on the environment and condition of use) * The product is not operated while charging. Charging time may be delayed, or charging may not be available depending on the type of charger. ※ It doesn't provide quick charging.

depending on the type of charger)

2 Unit ON

3 Unit ON

4 Unit ON

Connect USB to port to start charging.

the charging starts.

When red light on the power button is turned on,

X Time for charging: About 2.0 hours (It may vary)

3. When charging is completed, white light will be turned on.

こいつつしら